

Polycystic Liver Disease Panel

Test code: GA0201

Is ideal for patients with a clinical suspicion of isolated polycystic liver disease.

All the genes in the panel cause autosomal dominant disease, of which three genes associates to polycystic liver disease (*PRKCSH*, *SEC63* and *LRP5*) and two genes associates with autosomal dominant polycystic kidney disease (*PKD1* and *PKD2*).

About Polycystic Liver Disease

The Polycystic Liver Disease Panel covers isolated polycystic liver disease (PCLD) and autosomal dominant polycystic kidney disease (ADPKD) where hepatic cysts are present in over 80% of cases as the most frequent extra-renal manifestation. The disease is restricted to the liver in PCLD but ADPKD is a multi-system disorder with polycystic kidneys as the major feature. The most important non-cystic manifestations are vascular abnormalities including intracranial arterial aneurysms, aortic dilatation and dissection. Nearly half of all ADPKD patients have end-stage renal disease (ESRD) by age 60 years. The progressive growth of liver cysts is the most common extrarenal manifestation of ADPKD. The clinical spectrum of ADPKD is wide and variable even within the same family. The implicated gene has prognostic value as the median age at ESRD was 58 years for PKD1 and 79 years for PKD2 related disease. The clinical utility of genetic testing is estimated to be 42% in PCLD (PubMed: 20408955 and 24706814) and over 90% ADPKD (GeneReviews).

Availability

Results in 3-4 weeks

Gene set description

Genes in the Polycystic Liver Disease Panel and their clinical significance

Gene	Associated phenotypes	Inheritance	ClinVar	HGMD
GANAB	Polycystic kidney and/or polycystic liver disease 3	AD	7	12
<u>LRP5</u> *	Van Buchem disease, Osteoporosis-pseudoglioma syndrome, Hyperostosis, endosteal, Osteosclerosis, Exudative vitreoretinopathy, Osteopetrosis late-onset form type 1, LRP5 primary osteoporosis	AD/AR/Digenic	57	196
<u>PKD1</u> *	Polycystic kidney disease	AD	237	1923
PKD2	Polycystic kidney disease	AD	55	333
PRKCSH	Polycystic liver disease	AD	9	28
SEC63	Polycystic liver disease	AD	8	23

*Some regions of the gene are duplicated in the genome. [Read more.](#)

The gene has suboptimal coverage (means <90% of the gene's target nucleotides are covered at >20x with mapping quality score (MQ>20) reads), and/or the gene has exons listed under Test limitations section that are not included in the panel as they are not sufficiently covered with high quality sequence reads.

The sensitivity to detect variants may be limited in genes marked with an asterisk (*) or number sign (#)

Gene refers to the HGNC approved gene symbol; Inheritance refers to inheritance patterns such as autosomal dominant (AD), autosomal recessive (AR), mitochondrial (mi), X-linked (XL), X-linked dominant (XLD) and X-linked recessive (XLR); ClinVar refers to the number of variants in the gene classified as pathogenic or likely pathogenic in this database ([ClinVar](#)); HGMD

Blueprint Genetics

refers to the number of variants with possible disease association in the gene listed in Human Gene Mutation Database ([HGMD](#)). The list of associated, gene specific phenotypes are generated from [CGD](#) or Mitomap databases.

Non-coding disease causing variants covered by the panel

Gene	Genomic location HG19	HGVS	RefSeq	RS-number
PKD1	Chr16:2140209	c.12445-14T>C	NM_001009944.2	
PKD1	Chr16:2147825	c.10167+25_10167+43delGGCTGGGCTGGGGGTCCTG	NM_001009944.2	rs1197421698
PKD1	Chr16:2152273	c.9202-16G>A	NM_001009944.2	
PKD2	Chr4:88940551	c.596-59A>G	NM_000297.3	rs750504141

Test Strengths

Exons 1-33 of the *PKD1* have multiple segmentally duplicated pseudogenes that reduce sensitivity of NGS diagnostics in general. However, Blueprint Genetics custom assay has good coverage (>20x) with high mapping rates (mapping quality >40) for 99.5% of the target regions in *PKD1* gene. Our validation showed high mean coverage of 199X for the *PKD1* gene. Thus, our NGS Panel is not expected to have major limitations in detecting variants in *PKD1* gene although clinical validation has not been performed at large scale.

The strengths of this test include:

- CAP accredited laboratory
- CLIA-certified personnel performing clinical testing in a CLIA-certified laboratory
- Powerful sequencing technologies, advanced target enrichment methods and precision bioinformatics pipelines ensure superior analytical performance
- Careful construction of clinically effective and scientifically justified gene panels
- Some of the panels include the whole mitochondrial genome (please see the Panel Content section)
- Our Nucleus online portal providing transparent and easy access to quality and performance data at the patient level
- Our publicly available analytic validation demonstrating complete details of test performance
- ~2,000 non-coding disease causing variants in our clinical grade NGS assay for panels (please see 'Non-coding disease causing variants covered by this panel' in the Panel Content section)
- Our rigorous variant classification scheme
- Our systematic clinical interpretation workflow using proprietary software enabling accurate and traceable processing of NGS data
- Our comprehensive clinical statements

Test Limitations

Genes with partial, or whole gene, segmental duplications in the human genome are marked with an asterisk (*) if they overlap with the UCSC pseudogene regions. The technology may have limited sensitivity to detect variants in genes marked with these symbols (please see the Panel content table above).

This test does not detect the following:

- Complex inversions
- Gene conversions
- Balanced translocations
- Some of the panels include the whole mitochondrial genome but not all (please see the Panel Content section)
- Repeat expansion disorders unless specifically mentioned

- Non-coding variants deeper than ± 20 base pairs from exon-intron boundary unless otherwise indicated (please see above Panel Content / non-coding variants covered by the panel).

This test may not reliably detect the following:

- Low level mosaicism in nuclear genes (variant with a minor allele fraction of 14.6% is detected with 90% probability)
- Stretches of mononucleotide repeats
- Low level heteroplasmy in mtDNA (>90% are detected at 5% level)
- Indels larger than 50bp
- Single exon deletions or duplications
- Variants within pseudogene regions/duplicated segments
- Some disease causing variants present in mtDNA are not detectable from blood, thus post-mitotic tissue such as skeletal muscle may be required for establishing molecular diagnosis.

The sensitivity of this test may be reduced if DNA is extracted by a laboratory other than Blueprint Genetics.

For additional information, please refer to the Test performance section and see our Analytic Validation.

Test performance

The Blueprint Genetics polycystic liver disease panel covers classical genes associated with isolated polycystic liver disease and autosomal dominant polycystic kidney disease. The genes on the panel have been carefully selected based on scientific literature, mutation databases and our experience.

Our panels are sliced from our high-quality whole exome sequencing data. Please see our sequencing and detection performance table for different types of alterations at the whole exome level (Table).

Assays have been validated for different starting materials including EDTA-blood, isolated DNA (no FFPE), saliva and dry blood spots (filter card) and all provide high-quality results. The diagnostic yield varies substantially depending on the assay used, referring healthcare professional, hospital and country. Blueprint Genetics' Plus Analysis (Seq+Del/Dup) maximizes the chance to find a molecular genetic diagnosis for your patient although Sequence Analysis or Del/Dup Analysis may be a cost-effective first line test if your patient's phenotype is suggestive of a specific mutation type.

The genes on the panel have been carefully selected based on scientific literature, mutation databases and our experience.

Our panels are sectioned from our high-quality, clinical grade NGS assay. Please see our sequencing and detection performance table for details regarding our ability to detect different types of alterations (Table).

	Sensitivity % (TP/(TP+FN))	Specificity %
Single nucleotide variants	99.89% (99,153/99,266)	>99.9999%
Insertions, deletions and indels by sequence analysis		
1-10 bps	96.9% (7,563/7,806)	>99.9999%
11-50 bps	99.13% (2,524/2,546)	>99.9999%
Copy number variants (exon level dels/dups)		
1 exon level deletion (heterozygous)	100% (20/20)	NA
1 exon level deletion (homozygous)	100% (5/5)	NA
1 exon level deletion (het or homo)	100% (25/25)	NA
2-7 exon level deletion (het or homo)	100% (44/44)	NA

Blueprint Genetics



1-9 exon level duplication (het or homo)	75% (6/8)	NA
Simulated CNV detection		
5 exons level deletion/duplication	98.7%	100.00%
Microdeletion/-duplication sdrs (large CNVs, n=37)		
Size range (0.1-47 Mb)	100% (37/37)	

The performance presented above reached by Blueprint Genetics high-quality, clinical grade NGS sequencing assay with the following coverage metrics

Mean sequencing depth	143X
Nucleotides with >20x sequencing coverage (%)	99.86%

Performance of Blueprint Genetics Mitochondrial Sequencing Assay.

	Sensitivity % (TP/(TP+FN))	Specificity
ANALYTIC VALIDATION (NA samples; n=4)		
Single nucleotide variants		
Heteroplasmic (45-100%)	100.0% (50/50)	100.0%
Heteroplasmic (35-45%)	100.0% (87/87)	100.0%
Heteroplasmic (25-35%)	100.0% (73/73)	100.0%
Heteroplasmic (15-25%)	100.0% (77/77)	100.0%
Heteroplasmic (10-15%)	100.0% (74/74)	100.0%
Heteroplasmic (5-10%)	100.0% (3/3)	100.0%
Heteroplasmic (<5%)	50.0% (2/4)	100.0%
CLINICAL VALIDATION (n=76 samples)		
All types		
Single nucleotide variants n=2084 SNVs		
Heteroplasmic (45-100%)	100.0% (1940/1940)	100.0%
Heteroplasmic (35-45%)	100.0% (4/4)	100.0%
Heteroplasmic (25-35%)	100.0% (3/3)	100.0%
Heteroplasmic (15-25%)	100.0% (3/3)	100.0%
Heteroplasmic (10-15%)	100.0% (9/9)	100.0%
Heteroplasmic (5-10%)	92.3%(12/13)	99.98%



Blueprint Genetics



Heteroplasmic (<5%)	88.7% (47/53)	99.93%
Insertions and deletions by sequence analysis n=42 indels		
Heteroplasmic (45-100%) 1-10bp	100.0% (32/32)	100.0%
Heteroplasmic (5-45%) 1-10bp	100.0% (3/3)	100.0%
Heteroplasmic (<5%) 1-10bp	100.0% (5/5)	>0.9999
SIMULATION DATA /(mitomap mutations)		
Insertions, and deletions 1-24 bps by sequence analysis; n=17		
Homoplasmic (100%) 1-24bp	100.0% (17/17)	99.98%
Heteroplasmic (50%)	100.0% (17/17)	99.99%
Heteroplasmic (25%)	100.0% (17/17)	100.0%
Heteroplasmic (20%)	100.0% (17/17)	100.0%
Heteroplasmic (15%)	100.0% (17/17)	100.0%
Heteroplasmic (10%)	94.1% (16/17)	100.0%
Heteroplasmic (5%)	94.1% (16/17)	100.0%
Copy number variants (separate artificial mutations; n=1500)		
Homoplasmic (100%) 500 bp, 1kb, 5 kb	100.0%	100.0%
Heteroplasmic (50%) 500 bp, 1kb, 5 kb	100.0%	100.0%
Heteroplasmic (30%) 500 bp, 1kb, 5 kb	100.0%	100.0%
Heteroplasmic (20%) 500 bp, 1kb, 5 kb	99.7%	100.0%
Heteroplasmic (10%) 500 bp, 1kb, 5 kb	99.0%	100.0%
The performance presented above reached by following coverage metrics at assay level (n=66)		
	Mean of medians	Median of medians
Mean sequencing depth MQ0 (clinical)	18224X	17366X
Nucleotides with >1000x MQ0 sequencing coverage (%) (clinical)	100%	
rho zero cell line (=no mtDNA), mean sequencing depth	12X	

Bioinformatics

The target region for each gene includes coding exons and ± 20 base pairs from the exon-intron boundary. In addition, the panel includes non-coding variants if listed above (Non-coding variants covered by the panel). Some regions of the gene(s) may be removed from the panel if specifically mentioned in the ‘Test limitations’ section above. The sequencing data generated in our laboratory is analyzed with our proprietary data analysis and annotation pipeline, integrating state-of-the-art algorithms and industry-standard software solutions. Incorporation of rigorous quality control steps throughout the workflow of the pipeline ensures the consistency, validity and accuracy of results. Our pipeline is streamlined to maximize sensitivity



Blueprint Genetics



without sacrificing specificity. We have incorporated a number of reference population databases and mutation databases such as, but not limited to, [1000 Genomes Project](#), [gnomAD](#), [ClinVar](#) and [HGMD](#) into our [clinical interpretation software](#) to make the process effective and efficient. For missense variants, *in silico* variant prediction tools such as SIFT, PolyPhen, MutationTaster are used to assist with variant classification. Through our [online ordering and statement reporting system, Nucleus](#), the customer has an access to details of the analysis, including patient specific sequencing metrics, a [gene level coverage plot](#) and a [list of regions with inadequate coverage if present](#). This reflects our mission to build fully transparent diagnostics where customers have easy access to crucial details of the analysis process.

Clinical interpretation

We provide customers with the most comprehensive clinical report available on the market. Clinical interpretation requires a fundamental understanding of clinical genetics and genetic principles. At Blueprint Genetics, our PhD molecular geneticists, medical geneticists and clinical consultants prepare the clinical statement together by evaluating the identified variants in the context of the phenotypic information provided in the requisition form. Our goal is to provide clinically meaningful statements that are understandable for all medical professionals regardless of whether they have formal training in genetics.

Variant classification is the corner stone of clinical interpretation and resulting patient management decisions. Our classifications follow the [Blueprint Genetics Variant Classification Schemes](#) based on the [ACMG guideline 2015](#). Minor modifications were made to increase reproducibility of the variant classification and improve the clinical validity of the report. Our experience with tens of thousands of clinical cases analyzed at our laboratory allowed us to further develop the industry standard.

The final step in the analysis of sequence variants is confirmation of variants classified as pathogenic or likely pathogenic using bi-directional Sanger sequencing. Variant(s) fulfilling the following criteria are not Sanger confirmed: the variant quality score is above the internal threshold for a true positive call, and visual check-up of the variant at IGV is in-line with the variant call. Reported variants of uncertain significance are confirmed with bi-directional Sanger sequencing only if the quality score is below our internally defined quality score for true positive call. Reported copy number variations with a size <10 exons are confirmed by orthogonal methods such as qPCR if the specific CNV has been seen less than three times at Blueprint Genetics.

Our clinical statement includes tables for sequencing and copy number variants that include basic variant information (genomic coordinates, HGVS nomenclature, zygosity, allele frequencies, *in silico* predictions, OMIM phenotypes and classification of the variant). In addition, the statement includes detailed descriptions of the variant, gene and phenotype(s) including the role of the specific gene in human disease, the mutation profile, information about the gene's variation in population cohorts and detailed information about related phenotypes. We also provide links to the references used, congress abstracts and mutation variant databases used to help our customers further evaluate the reported findings if desired. The conclusion summarizes all of the existing information and provides our rationale for the classification of the variant.

Identification of pathogenic or likely pathogenic variants in dominant disorders or their combinations in different alleles in recessive disorders are considered molecular confirmation of the clinical diagnosis. In these cases, family member testing can be used for risk stratification within the family. In the case of variants of uncertain significance (VUS), we do not recommend family member risk stratification based on the VUS result. Furthermore, in the case of VUS, we do not recommend the use of genetic information in patient management or genetic counseling.

Our interpretation team analyzes millions of variants from thousands of individuals with rare diseases. Thus, our database, and our understanding of variants and related phenotypes, is growing by leaps and bounds. Our laboratory is therefore well positioned to re-classify previously reported variants as new information becomes available. If a variant previously reported by Blueprint Genetics is re-classified, our laboratory will issue a follow-up statement to the original ordering health care provider at no additional cost.

#}

ICD codes

Commonly used ICD-10 codes when ordering the Polycystic Liver Disease Panel

ICD-10	Disease
Q44.6	Isolated polycystic liver disease
Q61.2	Autosomal dominant polycystic kidney disease

Accepted sample types

- EDTA blood, min. 1 ml
- Purified DNA, min. 3µg*
- Saliva (Oragene DNA OG-500 kit)

Label the sample tube with your patient's name, date of birth and the date of sample collection.

Note that we do not accept DNA samples isolated from formalin-fixed paraffin-embedded (FFPE) tissue.

Resources

- [ARPKD/CHF Alliance](#)
- [British Liver Trust](#)
- [Canadian Liver Foundation](#)
- [Chebib FT et al. Effect of genotype on the severity and volume progression of polycystic liver disease in autosomal dominant polycystic kidney disease. Nephrol Dial Transplant. 2016 Jun;31\(6\):952-60.](#)
- [GeneReviews - Autosomal Dominant Polycystic Kidney Disease](#)
- [GeneReviews - Polycystic Kidney Disease](#)
- [NORD - Polycystic Liver Disease](#)
- [Polycystic Kidney Disease Charity](#)
- [Polycystic Kidney Disease Foundation](#)
- [Polycystic Liver Disease](#)
- [The National Kidney Foundation](#)
- [Wybrich R et al. Polycystic liver disease: an overview of pathogenesis, clinical manifestations and management. Orphanet J Rare Dis. 2014; 9: 69.](#)